



Linda-Sol® Hemorrhoids Barrier Spray

Hemorrhoids problem is a problem which has been faced for hundreds of years all around the world and many people are affected with this problem.

Linda-Sol which is manufactured by our company not only acts by a non-pharmacological way but also reduces the pain and the pain is not felt again.

It is expected to reduce itching even in the first use.

Starting from the second day of use, along with the reduction of blood clots in the hemorrhoid breast, the reduction in size starts. With the continued use of the product, day after day the reduction in the size of the hemorrhoid breast continues.

In Grade 1 and Grade 2 patients, 1 pack of 10ml bottle of Linda-Sol is good enough. Only if you are a Grade 3 or a Grade 4 patient, it will be required to use a 2nd bottle.

For more information and sales; email at info@esculinda.com

Purpose of usage:

It is intended to use in Hemorrhoid patients and in patients with anal fissure problems.

Indications:

The product forms a barrier at the anus and facilitates the product to stay for a long time in the area, protects the wounded area against microbial factors and works by eradicating the symptoms of hemorrhoids such as pain, itching and hemorrhoid breast.

Contraindications:

Persons who are allergic to the contents of the product and persons who develop allergy may not use the product.

Pregnant women should consult the doctor before use.

Method of use:

1. Remove the cap from the nozzle.
2. Spray several times in the air before initial use to fill up the nozzle.
3. Before use, look intently at the position of the spray. The spray can be used from any desirable angle.
4. Bend slightly forward, and if necessary, spread your buttocks in order to be able to reach the anus more easily.
5. Aim the opening of the spray at the anus and spray seven or eight times on and around the anus.
6. Put the cap back on the spray after use and wash your hands.

Duration of Use:

Must be used once every 24 hours. During the course of treatment, must be used everyday

Side Effects

No side effects have been found to exist during the studies

Linda-Sol is a Class IIa Medical Device.



ESCULINDA COMPANY WORLD HEADQUARTERS
10 FORTUNE STREET LANCASHIRE MANCHESTER BL3 2 QZ UNITED KINGDOM
+ 44 0 7481 618 298

sales@esculinda.com info@esculinda.com www.esculinda.com

